LEFT INTENTIONALLY BLANK

es

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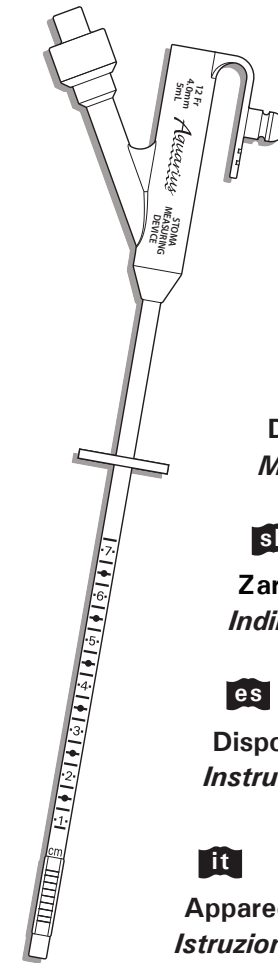
Aquarius™

en

Stoma Measuring Device Instructions for Use

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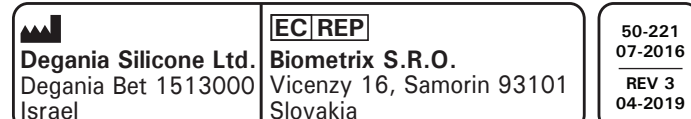
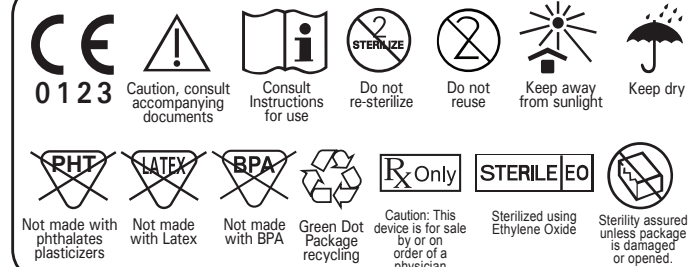
Device Description and Indication for Use

The **Aquarius™ Stoma Measuring Device** comprises a tubular shaft with graduated scale, a funnel with an inflation valve and a retaining balloon. The **Aquarius™ Stoma Measuring Device** is made of 100% silicone.

The **Aquarius™ Stoma Measuring Device** is designed to determine the depth (length) of a well established gastrostomy stoma tract in order to assist in proper length selection of the **Aquarius™ Gastrostomy button (G-Button)**. Intended for transient use (less than 60 minutes). **Aquarius™ Stoma Measuring Device** size: 12 Fr 5mL. **Never inflate more than 7mL.**

Aquarius™ G-Button Fr and Length Size(s)

| Length (mm) | 8 | 10 | 12 | 15 | 17 | 20 | 23 | 25 | 27 | 30 | 35 | 40 | 45 | 50 | 55 | 60 | 65 | | |
|--------------|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|--|--|
| Size (Fr/Ch) | 12 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | | |
| | 14 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | | |
| | 16 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | | |
| | 18 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | |
| | 20 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | |
| | 24 | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | | | | |

Read all instructions carefully before use!

Contraindications

Use of this device is contraindicated in individuals who do not have a well established stoma tract.

Warning

Each product is supplied in an individual sterile pouch. The product is guaranteed sterile unless the individual packaging has been opened or damaged.

For single use only. Do not reuse, reprocess or resterilize. Re-use may cause a risk of infection and/or compromise functional reliability. Do not use if the package is open or damaged. Verify expiration date. Do not use after this date.

Do not nick, cut, tear, puncture, or otherwise damage the device.

Do not use the device for enteral feeding.

If resistance is encountered while advancing the catheter through the stoma tract into the gastric lumen, this may indicate that the stoma has closed. Further attempts at forceful advancement of the catheter may result in the creation of a tract in the abdominal wall.

CAUTION:

Placement should be performed by, or under the direction of, the physician. Sizes (Fr and length) are to be prescribed by the physician

Placement of the Stoma Measuring Device

1. Lubricate the tip of the measuring device with water soluble lubricant. Do not use mineral oil. Do not use petroleum jelly. (Figure 1).

2. Gently slide the tip of the measuring device through the stoma site into the stomach. (Figure 2).

3. Use Luer Slip syringe to inflate the balloon with 5 mL of water and attach to the balloon port. Depress the syringe plunger and inflate the balloon.

4. Gently pull the device toward the abdomen until the balloon rests against the inside of the stomach wall. (Figure 3).

5. Slide the plastic disc down to the abdomen and record the measurement above the disc. (Figure 4).

6. Take two readings, one in the upright and one in the supine position. Calculate the average of the two readings and select the next larger size.

7. Use a slip-tip syringe to deflate the balloon before removing the **Aquarius™ Stoma Measuring Device**. (Figure 5).

8. Document the date, lot number and measured centimeter shaft length. The measurements are in 0.25 cm increments: every full centimeter is marked with a number.

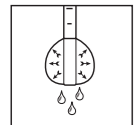
Figure 1

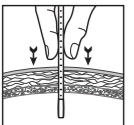
Figure 2

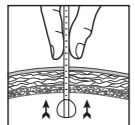
Figure 3

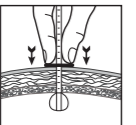
Figure 4

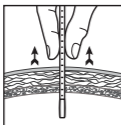
Figure 5

Warning

Under-sizing may cause erosion embedment in the gastric wall, tissue necrosis, infection, sepsis and associated sequelae.

Storage:

The products should be stored in the original packaging, at room temperature, and in a dry environment. Avoid direct sunlight and heat radiation.

* Aquarius™ is a registered trademark of Degania Silicone Ltd.

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* Aquarius™ est une marque déposée de Degania Silicone Ltd.